

## Commissioning hand-held remote control WC triggering MyMove

Before using the hand-held remote control for the first time, it must be paired with the existing control unit.

Please proceed as follows:

- De-energise the toilet system (control unit) for at least 1 minute and then re-energise it

You now have 1 minute to carry out the following actions:

- Use the existing MyMove remote control (white remote control) to move the toilet up or down and flush once.

For version with folding support handles:

- Then flush the toilet again using the toilet flush trigger (blue remote control) on the toilet



Now wait two minutes and test all functions (raising and lowering) again.

Important note:

Please ensure that there are no other MyMove systems in operation in the room where the commissioning is taking place.

If there are several MyMove systems in a room, the systems that are not to be connected to the remote control must be de-energised at the time of commissioning.

## Technische Details my move WC care

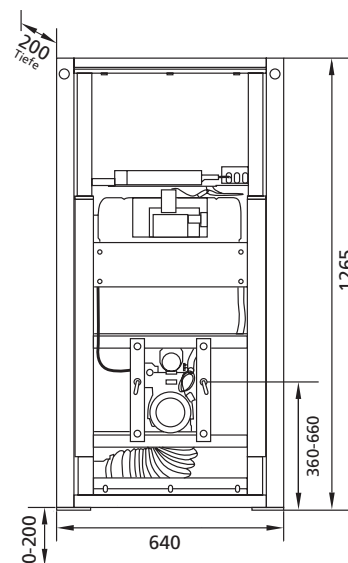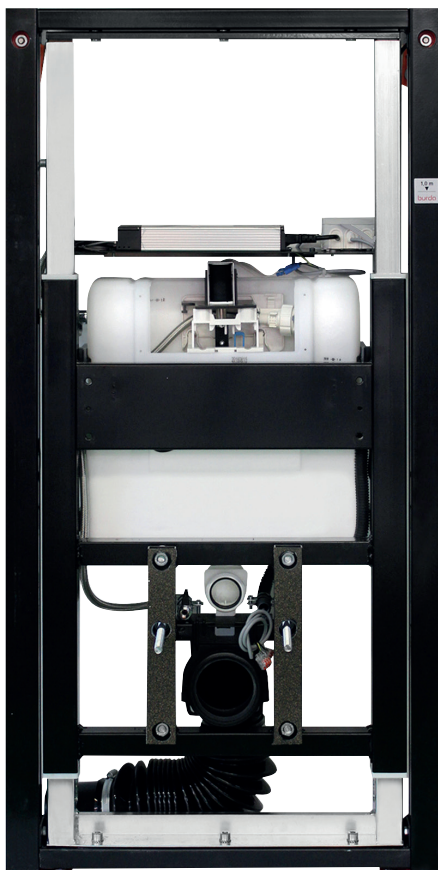

### Optional gegen Aufpreis lieferbar:

- |  |        |
|--|--------|
| • Individuelle Glasplattenfarbe nach RAL                   | 270003 |
| • Verkleidung für die Einzelmontage, RAL Classic-Farben    | 270005 |
| • Verkleidung für die Einzelmontage, RAL Design-Farben     | 270007 |
| • Vorbereitung für Aufnahme von Dusch-WC                   | 270010 |
| • Externer Taster "hoch/runter"                            | 270008 |
| • Externer Taster "spülen"                                 | 270009 |
| • Rückenstütze   | 270012 |
| • Zweiter Papierrollenhalter                               | 270013 |
| • Zweite Steuerung am Stützklappgriff (hoch/runter/spülen) | 270014 |